## XL Calibrant B

Cat. No.	Pack Name	Packaging (Content)
XSYS0065	XL CAL B	1 x 52 ml

EN

### INTENDED USE:

Calibrant B is a buffered saline reagent used for 2 point calibration of Sodium, Potassium and Chloride electrolytes in the ISE module of the XL Clinical Chemistry Auto-analyzer.

**WARNING:** For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Reagent of the kit is not classified as dangerous.

**REAGENT PREPARATION:** The reagent is ready to use.

REAGENT COMPOSITION:	Active Ingredients	Concentration
	Sodium Chloride	70 mmol/l
	Potassium Chloride	8 mmol/l
	Triton - X 100	< 0.01%
	Buffers, Preservatives	< 0.04%

### REAGENT STORAGE & STABILITY:

Store reagents at room temperature (15...25°C). The reagents are stable till the expiry date mentioned on the label.

### PROCEDURE:

A quantity of approximately 500 µl is to be placed in the XL sample cup labeled Cal B (Position ISE 1 on the Standard tray) of the XL analyzer, at the time of performing an ISE calibration.

**PRECAUTIONS:** 1. Avoid contaminating the reagent.  
2. Keep the container tightly closed and store the reagent in the original container.

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