XL Calibrant B

Cat. No.	Pack Name	Packaging (Content)
XSYS0065	XL CAL B	1 x 52 ml

INTENDED USE: Calibrant B is a buffered saline reagent used for 2 point calibration of Sodium, Potassium and Chloride electrolytes in the ISE module of the XL Clinical Chemistry Auto-analyzer.

WARNING: For *in vitro* diagnostic use. To be handled by entitled and professionally educated person. Reagent of the kit is not classified as dangerous.

REAGENT PREPARATION: The reagent is ready to use.

REAGENT COMPOSITION: Active Ingredients	Concentration
Sodium Chloride	70 mmol/l
Potassium Chloride	8 mmol/l
Triton - X 100	< 0.01%
Buffers, Preservatives	< 0.04%

REAGENT STORAGE & STABILITY: Store reagents at room temperature (15...25°C). The reagents are stable till the expiry date mentioned on the label.

PROCEDURE: A quantity of approximately 500 µl is to be placed in the XL sample cup labeled Cal B (Position ISE 1 on the Standard tray) of the XL analyzer, at the time of performing an ISE calibration.

PRECAUTIONS: 1. Avoid contaminating the reagent.

2. Keep the container tightly closed and store the reagent in the original container.

SYMBOLS: The following symbols are used in the labelling of ERBA kits:

 REF	Catalogue Number	 Manufacturer	 See Instruction for Use
 LOT	Lot Number	 CE Mark - Device comply with the Directive 98/79/EC	 Storage Temperature
 Expiry Date		 In Vitro Diagnostics	 Content

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